


ALWAYS GREEN BEFORE YOU GO!




M210® Control Unit

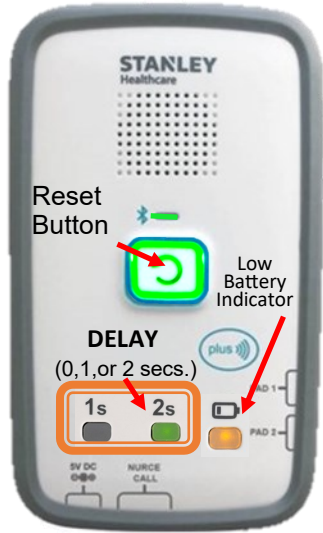


Side View - Sensor Pad Ports




Sensor Pad PAD1 and PAD2 ports.

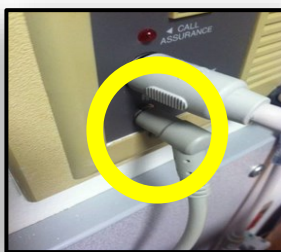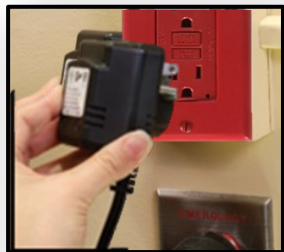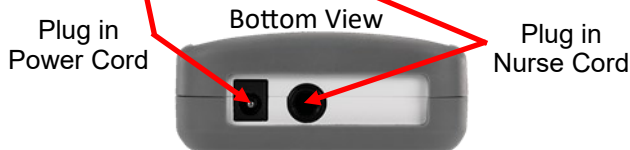

Mount M210 on head wall in patient room

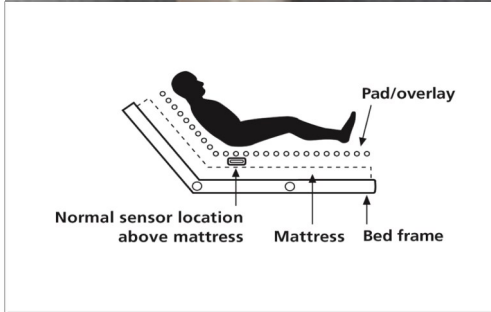

BED: Place sensor pad under the sheets in the hip area. Run sensor pad cord along mattress towards M210. Plug sensor pad into the M210.

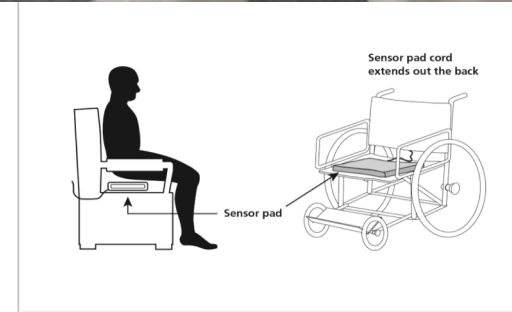

CHAIR: Place sensor pad across the width of the seat; place under hip area.

PORTABLE OPTIONS



WHEELCHAIR



TOILETING

NOTE: 2 sensor pads or 1 sensor pad & 1 lap belt can be plugged in at the same time but **cannot** be monitored simultaneously.

NOTE: Batteries are not rechargeable.

ALWAYS CHECK GREEN MONITORING BEFORE YOU GO!

**GREEN
MEANS
GO**

**Always verify green
monitoring light is on
before leaving patient.**



**Green = Patient is
being monitored.**



**SILENCE
ALARM**

**Press RESET for 1 second
to silence M210 alarm**



**If silencing M210 alarm
does not cancel nurse
call; cancel nurse call if
applicable.**

**TO
EXIT
PATIENT**

HOLD Mode:

To place M210 on HOLD: Push
RESET and hold for 3 seconds until
you hear "HOLD MODE"
confirmation.


- 30 seconds to exit patient then
M210 will be on standby.



**TO
ALLOW
REPOSITIONING**

**Press the RESET twice to
change DELAY**

DELAY allows patient to move & shift
on the sensor pad without alarm
sounding.

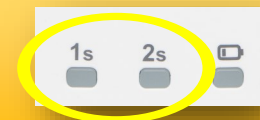

Setting Options: 0, 1, 2

Note: When set to 0 seconds, no
icon will be lit on front of M200