

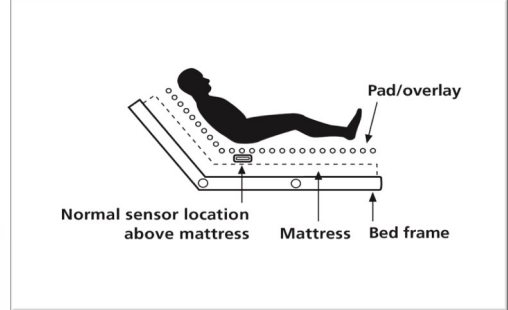
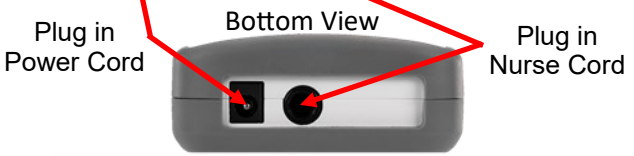


ALWAYS GREEN BEFORE YOU GO!

M210® Control Unit

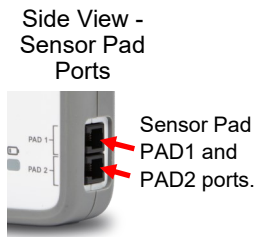

BED: Place sensor pad under the sheets in the hip area. Run sensor pad cord under mattress towards M210. Plug sensor pad into the M210.

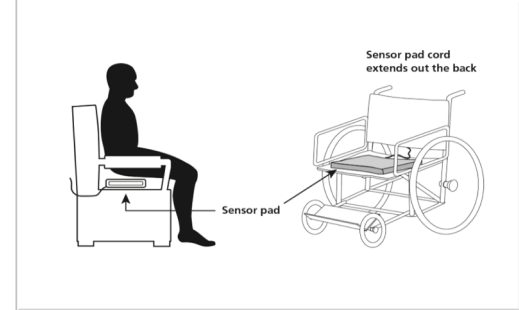

CHAIR: Place sensor pad across the width of the seat; place under hip area.

PORTABLE OPTIONS



WHEELCHAIR



TOILETING

NOTE: 2 sensor pads or 1 sensor pad & 1 lap belt can be plugged in at the same time but **cannot** be monitored simultaneously.
NOTE: Batteries are not rechargeable.

ALWAYS CHECK GREEN MONITORING BEFORE YOU GO!

**GREEN
MEANS
GO**

**Always verify green
monitoring light is on
before leaving patient.**



**Green = Patient is
being monitored.**



**SILENCE
ALARM**

**Press RESET for 1 second
to silence M210 alarm**



If silencing M210 alarm
does not cancel nurse
call; cancel nurse call if
applicable.

**TO
EXIT
PATIENT**

HOLD Mode:

To place M210 on HOLD:
Push RESET for 3 seconds and you
hear audible confirmation.

- 30 seconds to exit patient.
- Now M210 is on standby.




SUSPEND Mode:

Push RESET until you hear
audible "SUSPEND MODE"
confirmation

**TO
ALLOW
REPOSITIONING**

**Press the RESET twice to
change DELAY**

DELAY allows patient to move & shift
on the sensor pad without alarm
sounding.

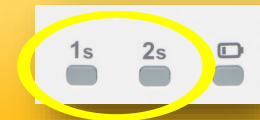

Setting Options: 0, 1, 2

Note: When set to 0 seconds, no
icon will be lit on front of M210