

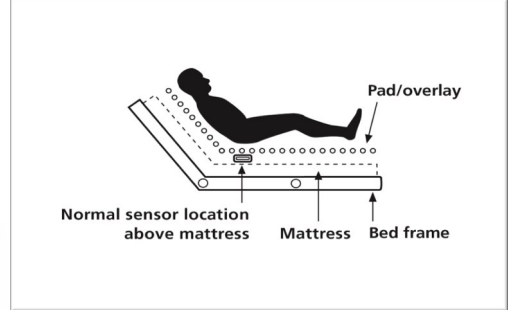
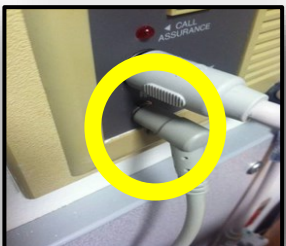
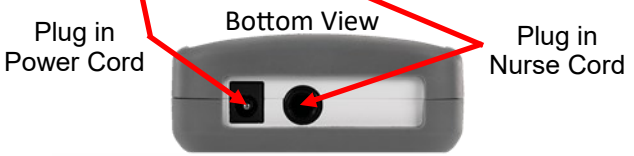


**GREEN MEANS YOU ARE GOOD TO GO!**

## M210® Control Unit

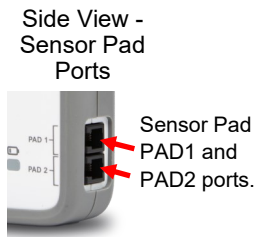

**BED:** Place sensor pad under the sheets in the hip area. Run sensor pad cord under mattress towards M210. Plug sensor pad into the M210.

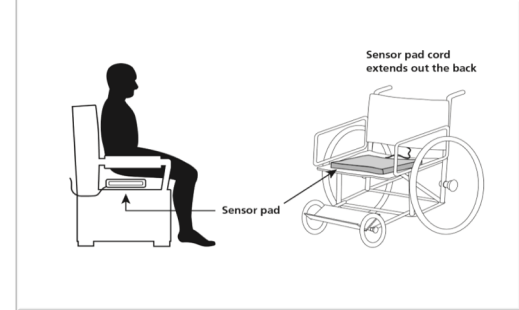

**CHAIR:** Place sensor pad across the width of the seat; place under hip area.

### PORTABLE WHEEL-CHAIR OPTION



**NOTE:** 2 sensor pads can be plugged in at the same time but **cannot** be monitored simultaneously.  
**NOTE:** Batteries are not rechargeable.

**ALWAYS CHECK GREEN MONITORING BEFORE YOU GO!**

**GREEN =  
GOOD TO  
GO**

**Always verify green  
monitoring light is on  
before leaving patient.**



**Green = Patient is  
being monitored.**



**SILENCE  
ALARM**

**Press RESET for 1 second  
to silence M210 alarm**



If silencing M210 alarm  
does not cancel nurse  
call; cancel nurse call if  
applicable.

**TO MOVE  
PATIENT**

### **HOLD Mode:**

To place M210 on HOLD:  
Push RESET for 3 seconds and you  
hear audible confirmation.

- 30 seconds to exit patient.
- Now M210 is on standby.



### **SUSPEND Mode:**

Push RESET until you hear  
audible "SUSPEND MODE"  
confirmation

**GREEN MEANS YOU ARE GOOD TO GO,  
YOUR PATIENT IS BEING MONITORED!**